

Short Instruction for Programming

1	P	Start of the Program " PR NR " appears on the display
2	1 to 30	Choose a program number on the numeric keypad
3	ENTER	" Set p " appears on the display
4	A <input type="text"/> to F <input type="text"/>	Press A to F to select a pump " P-X " appears on the display
5	XX	Please use the numeric keypad to enter the quantity of liquid (ml/100g powder)
6	ENTER	" Pr-X " appears on the display
7	ENTER or 1.XX	Please indicate the density or confirm with " ENTER " the density of 1
8	ENTER	" Set P " appears on the display
9	ENTER or to 4	You can store the program when pressing " ENTER " or you can go further in programming starting with point 4 again

Instruction to Start a Program

1	START	Start of the program " PR NR " appears on the display
2	1 to 30	Choose a program number on the numeric keypad
3	ENTER	To confirm the program no.